

Reports Overview
Connect-Rx Version 8.0



AcuDose-Rx Reporting

Improve Compliance, System Utilization, and Inventory Management

The all new AcuDose-Rx® automated dispensing cabinet offers the most robust reporting available. Data is consolidated from all AcuDose-Rx cabinet locations and other McKesson automation devices on the integrated Connect-Rx® system platform.

- **Compliance Reporting.** Reports to help you comply with HIPAA, Joint Commission, and other regulatory standards.
- **System Utilization Reporting.** Reports to help you get the most out of your system assets—both people and technology.
- **Inventory Management Reporting.** Reports to help you improve charge capture and medication inventory management.

Push-button Reporting

With AcuDose-Rx 8.0, reporting just got even simpler. More than 70 standard reports are ready to run out of the box from any networked station.

- Be more efficient because you can run, modify, save, and print reports from any networked PC.
- Save time by re-running common reports without re-entering all parameters.
- Quickly identify key metrics when you multi-select associated values and parameters, such as multiple medications or nursing units.
- Use easier to understand terms (“last 7 days,” “yesterday,” etc.) to quickly enter dates.
- Share more easily with colleagues by saving in PDF, MS Word, Excel or other common formats.

Comprehensive Reporting

Standard Reports

Using the intuitive and highly popular Crystal Reports format, the more than 70 standard AcuDose-Rx reports help monitor, track, analyze, and hold staff accountable in the performance of a hospital's medication-use system. Authorized users can access, view, and print reports through any AcuDose-Rx main cabinet, or through any Connect-Rx system workstation. For a more detailed description of all standard AcuDose-Rx reports, please consult the AcuDose-Rx Reports Manual.

Open Database Access

Users can develop their own custom reports by extracting data from the database. This avoids the need to request custom reports.



Narcotic diversion and enterprise-wide inventory management are just a few of the popular uses for the stand-alone Automation Decision Support business intelligence tool.

ACUDOSE-RX OPTIMIZATION REPORT		
FACILITY: REHAB - McKesson Rehab		
UNIT: ER2 - Emergency Room 2		
STATION: ER2 - Narc2 Filling Cabinet (*)		
ACARBORSE 50 MG TAB		
Max Orders/No: Avg Daily Use: 0.0		
Location	Packet Type	Packet Capacity
1-2-3	Locked/Loaded double	30
1-3-4	Locked/Loaded single	40
		70
ACETAMINOPHEN 80 MG TAB CHEW		
Max Orders/No: Avg Daily Use: 0.0		
Location	Packet Type	Packet Capacity
1-3-5	Locked/Loaded double	0
ACETAMINOPHEN/CAFFEINE/BUTALB 325-40-50 TAB		
Max Orders/No: Avg Daily Use: 0.0		
Location	Packet Type	Packet Capacity
1-3-7	Locked/Loaded double	30
ALOE VERA POWDER		
Max Orders/No: Avg Daily Use: 0.0		
Location	Packet Type	Packet Capacity
1-4-2	Locked/Loaded single	0
BACITRACIN 500 UNIT/G OINTMENT		
Max Orders/No: Avg Daily Use: 0.0		
Location	Packet Type	Packet Capacity
1-3-3	Locked/Loaded double	0
BACLOFEN 10 MG TAB		
Max Orders/No: Avg Daily Use: 32.0		
Location	Packet Type	Packet Capacity
1-1-1	Locked/Loaded single	0
GABAPENTIN 400 MG CAPSULE		
Max Orders/No: Avg Daily Use: 0.0		
Location	Packet Type	Packet Capacity
1-1-4*	Locked/Loaded single	0
*Pending Medication Assignment **Max gran		
McKesson Hospital		
09/16/2010 9:21 Refill Method: A		

ACUDOSE-RX OUT OF STOCK POCKETS						
Count of Stockouts is within last 14 days						
Number of stockout days = 14						
Product = AcuDose-Rx						
FACILITY: MAIN - McKesson Main Hospital						
STATION: anth1 - anth one						
Location	Medication Description	Max	Per	Packet Capacity	Last Stockout	# of Stockouts
1-2-1	ACETAMINOPHEN 325 MG TAB	10	3	0	0	0
1-2-4	ACETAMINOPHEN/CAFFEINE 500-32MG TAB	10	3	0	0	0
1-3-8	IBUPROFEN 800 MG TAB	10	3	0	0	0
1-4-8	IBUPROFEN 400 MG TAB(BUSMG)	10	3	0	0	0
STATION: PERF14 - Performance 14						
Location	Medication Description	Max	Per	Packet Capacity	Last Stockout	# of Stockouts
1-1-2	IBUPROFEN 200 MG TAB	16	15	0	0	0
1-1-3	ASPIRIN 81 MG TAB	4	3	0	0	0
1-1-4	DOXUSATE SODIUM 100 MG CAP	16	15	0	0	0
1-2-2	ABACAVIR SULFATE 300 MG TAB	16	15	0	0	0
1-2-3	ACAMPROSTATE CALCIUM 333 MG TAB DS PK	16	15	0	0	0
1-2-4	ACETAMINOPHEN 325 MG TAB	16	15	0	0	0
1-3-4	ACYCLOVIR 800 MG TAB	16	15	0	0	0
1-4-1	ACEFENIVIR OPHTHAL 10 MG TAB	16	15	0	0	0
1-4-2	ALBUTEROL 80MG PUFFS	16	15	0	0	0
1-4-3	ALENDRONATE SODIUM 10 MG TAB	16	15	0	0	0
1-4-4	ALFALFA TAB	16	15	0	0	0
1-5-1	ALLOPURINOL 300 MG TAB	16	15	0	0	0
1-5-2	ALMOTRIPTAN MALATE 12.5 MG TAB	16	15	0	0	0
1-5-3	ALOSETRON HCL 0.5 MG TAB	16	15	0	0	0
1-5-4	ALPRAZOLAM 0.25 MG TAB	16	15	0	0	0
1-6-1	AMBENONIUM CHLORIDE 10 MG TAB	11	10	0	0	0
1-6-2	2-DEOXY-D-GLUCOSE POWDER	16	15	0	0	0
1-6-3	A-ACETYL MANDELIC ACID 100% POWDER	4	3	0	0	0
1-6-4	AMOXICILIN TRIHYDRATE 500 MG CAPSULE	15	14	0	0	0
1-7-1	DIOSMIN 120 MG TAB	16	15	0	0	0
1-7-2	FINASTODINE 40 MG TAB	12	11	0	0	0
1-7-3	IBUPROFEN 800 MG TAB	16	15	0	0	0
STATION: PERF25 - Perf25 DTL						
Location	Medication Description	Max	Per	Packet Capacity	Last Stockout	# of Stockouts
1-1-2	IBUPROFEN 200 MG TAB	16	15	0	0	0
1-1-3	DOXUSATE SODIUM 100 MG CAP	16	15	0	0	0
1-1-4	ACETAMINOPHEN 500 MG TABLET	4	3	0	0	0
1-1-6	ABACAVIR SULFATE 300 MG TAB	16	15	0	0	0
1-1-7	ACAMPROSTATE CALCIUM 333 MG TAB DS PK	16	15	0	0	0
1-1-8	ACETAMINOPHEN 325 MG TAB	16	15	0	0	0
1-1-12	ACYCLOVIR 800 MG TAB	16	15	0	0	0
McKesson Hospital						
11/29/2010 14:44						
Version 8.0 Rev 5						
Reviewed: 09/17/2010 14:13						
Generated By: MAH SERVICE						
Page 1 of 5						

Among the new reports available with AcuDose-Rx, are the *Optimization by Station Report* (top) and the *Out of Stock Pockets Report*. Both reports are tools of the RightStock™ system, the dynamic inventory optimization feature set exclusively available with AcuDose-Rx cabinets.

Automation Decision Support

The optional Automation Decision Support™ solution compiles data from AcuDose-Rx cabinets—plus other McKesson Automation solutions—to focus management attention and make faster, better-informed strategic decisions.

Hospitals leaders use this business intelligence tool to improve productivity and performance through enhanced monitoring of diversion, visibility into inventory levels, tracking of daily dispensing performance, and more.

For example, hospital leaders can examine different dispensing patterns, including unauthorized openings of all drawers and pockets from AcuDose-Rx medication dispensing cabinets to identify potential drug diversion issues.

This advanced system monitors performance of automation systems across a single site, several hospitals, or a multi-facility enterprise. Customizable views—such as dashboards, scorecards, tables, graphs, and three-dimensional pie charts—enable managers to get a comprehensive overview.

MCKESSON

McKesson Automation Solutions

500 Cranberry Woods Drive
 Cranberry Township, PA 16066
 www.mckesson.com
 (724) 741-8000

Copyright © 2011 McKesson Corporation and/or one of its subsidiaries. McKesson Corporation product and service names listed herein which bear the ® and ™ symbols are registered with the U.S. Patent and Trademark Office. All other product and service names are protected by non-registered trademarks or service marks, respectively.